

# Sheep50K



**Animal Health**  
**Animal Genetics**

PO Box 5520  
Dunedin, New Zealand  
Ph: 0800 228 278  
Fax: +64 3 4775 930  
pahgenetics.nz@pfizer.com  
www.pfizeranimalgenetics.co.nz

## Sample Collection for Sheep50K Testing

Sheep50K is a DNA based laboratory test. This test requires a whole blood tube to be taken from animals to ensure adequate concentrations of DNA are received.

We require no less than 3mL of Whole blood per animal to be submitted in **ONLY purple or green** topped tubes.

After collection, the samples should be clearly labelled with the animal's identification details and refrigerated until they are able to be couriered to the office of Pfizer Animal Genetics, at the below address::

**Pfizer Animal Genetics**

Level 4  
9 Moray Place  
Dunedin 9016  
Phone 03 477 5920

The samples need to be couriered in cold storage (an ice pack in a Styrofoam box will suffice) along with a signed sample submission form and a signed copy of the terms and conditions, both available from Customer Services (0800 228 278) or by download from:

[http://www.pfizeranimalgenetics.co.nz/sites/PAG/nz/Pages/Testing\\_and\\_Results.aspx](http://www.pfizeranimalgenetics.co.nz/sites/PAG/nz/Pages/Testing_and_Results.aspx)

Pfizer Animal Genetics can contribute to the cost of sample collection once per year per customer, whether the customer is an individual or part of a group. Receipt of this contribution is not automatic, and customers must liaise with Pfizer Animal Genetics to receive it.

Should you have any questions please don't hesitate to contact us on 0800 228 278.

Kind Regards,

Pfizer Animal Genetics

# Sheep50K

For more information please contact **Pfizer Animal Genetics**

P 0800 228 278

P 64 3 477 5920

F 64 3 477 5930

E [pahgenetics.nz@pfizer.com](mailto:pahgenetics.nz@pfizer.com)

W [www.pfizeranimalgenetics.co.nz](http://www.pfizeranimalgenetics.co.nz)

This technology is a result of New Zealand farmer investment in **Beef + Lamb** New Zealand and Ovita Ltd.